



3shape 

3Shape TRIOS®

Intraoral Workflow Guide



800-320-0230 * www.bayoustatelab.com

How to Fill Out an Rx



Select appointment time



1. Select Bayou State Crown & Bridge Lab Inc.
2. Fill out the patient information, fill out implant information
3. Select the scanner icon



800-320-0230 * www.bayoustatelab.com

How to Prep Scan



1. Select "Prep Arch"
2. Begin scanning



When scan is complete, press tooth #19 icon then press the scan abutment



Scan abutment indicated



1. Select scan body icon
2. Void will appear on the model
3. Once again, scan void area – this will be in high resolution



800-320-0230 * www.bayoustatelab.com

How to Prep Scan



When scan is complete, press #19 icon then press the scan abutment



Scan abutment indicated



Select "Upper"



Flip the scan tip over 180° then scan the opposing arch




How to Prep Scan



1. Select "Scan Occlusion"
2. Once again, flip the scan tip over to its original position



Scan the bite

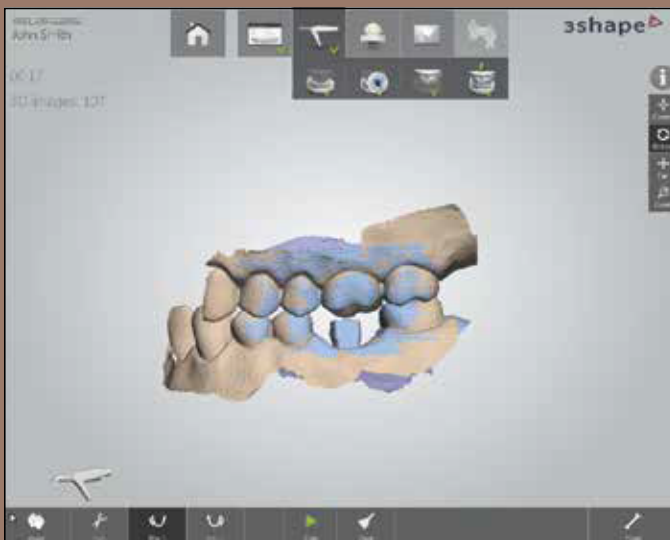


Bite has been established



800-320-0230 * www.bayoustatelab.com


How to Send Your 3Shape Files to Bayou State Lab

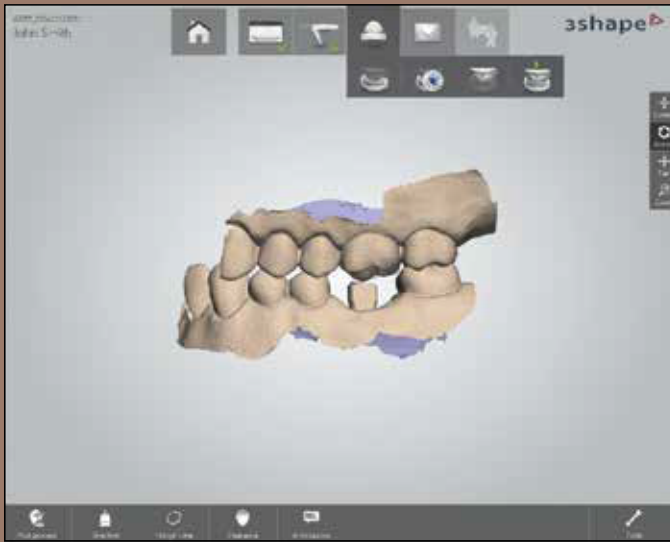

1. Press tooth icon
2. Press "Post Process"



Processing scan data

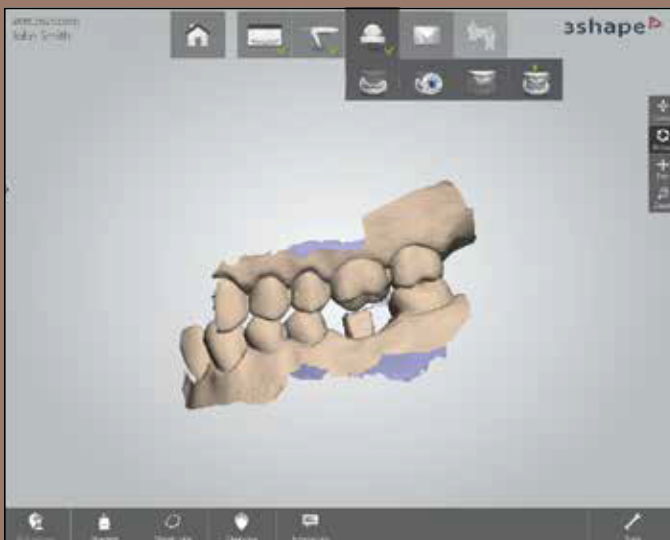

Press "Mail"



800-320-0230 * www.bayoustatelab.com

How to Send Your 3Shape Files to Bayou State Lab



The screenshot shows the 3shape software interface. At the top, there is a navigation bar with icons for home, printer, scanner, user, mail, and a 3shape logo. Below this, a 'Send order' form is displayed. The form contains the following fields:

Send order	
Patient	John Smith
Scheduled	4/11/2013 10:00 AM
Items	Removable Crown LP
Materials	
Color	A3
Notes	Lower, Lower Right, Upper, Dentures aligned
Lab	Bayou State Lab

At the bottom of the form, there is a 'Send order' button with an envelope icon.

Press "Send Order"



Order successfully sent



800-320-0230 * www.bayoustatelab.com

For more information give us a call 800-320-0230



BAYOUSTATE

CROWN & BRIDGE LAB

WWW.BAYOUSTATELAB.COM

800-320-0230 * www.bayoustatelab.com